GB US

IceBand® is a cooling and compression wrap that has been specifically developed for the relief of postoperative pain and swelling after foot/ankle surgery.

Reading this Users' Manual will enable you to obtain the maximum therapeutic effect.

IceBand® should be applied without delay after surgery and can also be used after physical therapy and rehabilitation.

IceBand® can facilitate postoperative rehabilitation and shorten your recovery.

Cooling the foot/ankle has an anti-inflammatory effect that can reduce swelling and pain.

IceBand® can also be used to help reduce pain and swelling following an acute injury, while you are awaiting medical attention.

IceBand® includes 1 foot/ankle wrap and 2 sets of 2 cooling elements. The cooling agent is a gel named soft ice and is non toxic.

USERS' MANUAL

- Put the 4 cooling elements in a freezer (within approx. 3 hours they will be frozen).
- Insert one cooling element in each of the two pockets of the wrap and apply the wrap to your foot. Keep the remaining cooling elements in the freezer and use these as replacements when the first two have melted.
- Tighten the Velcro fastenings securely to provide good compression and cooling.
- Allow the IceBand® to work for 45-60 minutes for maximum effect. We recommend using IceBand for repeated intervals of approx. 45-60 minutes each time. After the 45-60 minutes, take off the IceBand for at least one hour. This enables the operated area to "rest". Repeat the process several times.
- Repeat the treatment several times postoperatively and after physical therapy and rehabilitation sessions as instructed by your healthcare professional.

The cooling elements shall never be applied directly to the skin. Only to be used inside the wrap.

CAUTION! Patients' skin should be observed frequently due to individual sensitivity and susceptibility to cool.

Contact your physician if the treatments lead to more pain or if you are uncertain about any aspect of its use. Cooling products shall be handled with care and only be used in accordance with instructions from a physician.

IceBand® should not be used if you suffer from cold allergy, Raynauds phenomenon or cryoglobulinaemia. If you are uncertain please ask your physician.

IceBand® was developed in co-operation with: Robert Ihrman, MD, Orthopaedic Consultant, Orthopaedic Clinic, Ryhov County Hospital in Sweden.

For single patient use only.

Each IceBand® can be used approximately 30 times.

CLEANING: The cooling elements can be disinfected using a surface disinfectant and can be washed in a dishwasher at max. 65 degrees Celsius / 150 Fahrenheit. The fabric shall not be washed.

DISPOSAL: Consists of water-based gel, polyethylene, and polypropylene and can be exposed of as normal household waste. When incinerated it will turn into CO² and water.

CE class 1
Non sterile
Producer: IB Medical AB, Sweden

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All rights reserved
IceBand® is a registered trademark
IceBand® is a registered trademark owned by IB MEDICAL AB
Hersteller: IB Medical AB, Schweden